SafeMask[®] Master Series

ASTM 3



DETAILED MASK DIMENSIONS						
Height of Earloop	63,5 mm					
Pleat Depth	13 mm					
Length of Nose-piece	120 mm					





TECHNICAL DATA SHEET

FEATURES & BENEFITS

- A beautiful line of masks designed in a variety of unique and vibrant colours so that your mask is as personalized as the dental care you provide.
- o Master Series ASTM Masks are engineered with Simply Soft Technology™ for a smooth, luxurious and comfortable fit, a perfect blend of breathability and softness.
- Soft, strong, ultrasonically sealed earloops attached to outside of mask prevent pulling and irritation and allow easy removal with less risk of self-contamination.
- o Shingle pleat design prevents fluid pooling and cross-contamination.
- o High-quality media ensures optimal filtration and breathability.

RECOMMENDED USE

 Procedure masks are intended to be worn by personnel during medical procedures to protect both the patient and the operating personnel from transfer of microorganisms, body fluids and particulate material transfer. Latex free.

PRODUCT DETAILS

Product Name: Medicom SafeMask® Master Series

Procedure Earloop Face Mask

Product Type: Single-use, non-sterile, three-layer construction


Earloop: Polyester and polyurethane


Nosepiece: Aluminum
Colour: See Appendix 1

Quantity per Inner Carton: 50 Country of Origin: USA

ITEM	COLOUR DIMENSIONS
	Ocean Surf 176 (+/- 5 mm)
2057	89 (+/- 3 mm)

ASTM F2100 - LEVEL 3 BARRIER PROTECTION						
PERFORMANCE FACTORS	TEST METHODS	RESULT				
Bacterial filtration efficiency (BFE)	ASTM F2101	≥ 98%				
Sub-micron particulate efficiency at 0.1µm (PFE) /	ASTM F2299	≥ 98%				
Resistant to penetration by synthetic blood	ASTM F1862	@ 160 mm Hg				
Differential Pressure (ΔP)	EN 14683	< 6.0 mm H ₂ O/cm ²				
Flame Spread	16 CF 1610	Class 1				

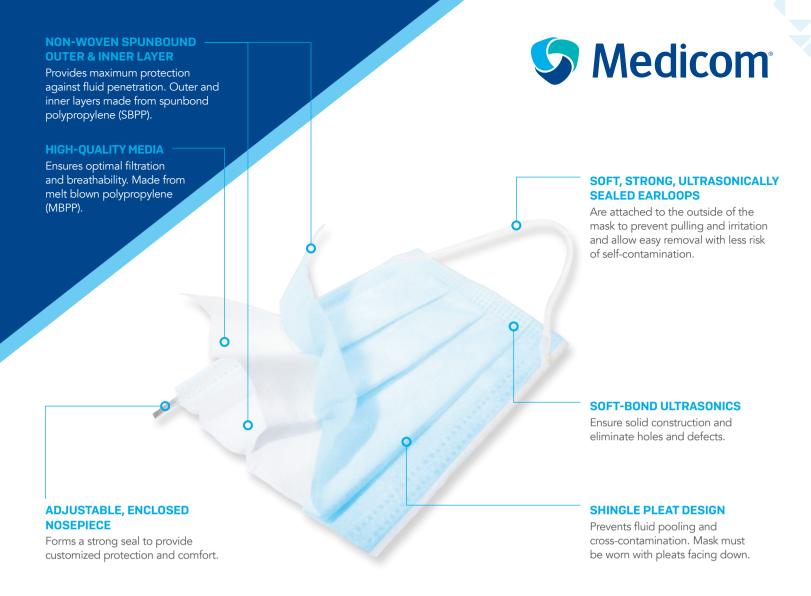

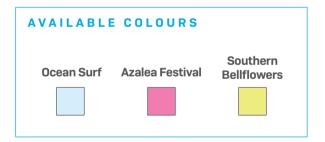

CERTIFICATION & STANDARDS

ISO 13485 certified manufacturer. Class 1 Medical Device. Health Canada registered medical device manufacturer.

STORAGE CONDITIONS

These masks should be disposed of after each use. Store in a cool, dry place. Avoid excessive heat (40 °C or 104 °F).





DONNING INSTRUCTIONS

- o Fan to open pleats.
- o Shape before applying to ensure a better fit.
- o Pull earloops over each ear.
- With two fingers on either side of your nose, firmly shape around your nose and across your cheeks, concentrating on the bridge of your nose, to secure the seal.
- Pinch the nosepiece and make sure the base of the mask sits comfortably below your chin.

DOFFING INSTRUCTIONS

- o Pull the earloops off your ears.
- Pull the mask away from your face, taking care not to touch its outer surface.
- Still holding only the earloops and taking care not to touch outer surface, dispose of mask in a clinical waste bin.
- o Always wash your hands thoroughly after removing a mask.



	Appendix 1 (List of items)									
Item Code	Description	Packaging Information	Color	Specific Attribute	Treatment	Marketed in USA as	Marketed in Canada as			
2057	Medicom SafeMask Master Series Procedure Earloop Face Masks	50 Masks/Box, 10 Boxes/Case, 500 Masks/Case	Ocean Surf	Aluminum nosepiece, Polyester & Polyurethane earloop, Flat-knitted earloop; "Medicom" embossed at the bottom of mask	Non-Sterile	Medical Device Class 2	Medical Device Class 1			
2058	Medicom SafeMask Master Series Procedure Earloop Face Masks	50 Masks/Box, 10 Boxes/Case, 500 Masks/Case	Azalea Festival	Aluminum nosepiece, Polyester & Polyurethane earloop, Flat-knitted earloop; "Medicom" embossed at the bottom of mask	Non-Sterile	Medical Device Class 2	Medical Device Class 1			
2059	Medicom SafeMask Master Series Procedure Earloop Face Masks	50 Masks/Box, 10 Boxes/Case, 500 Masks/Case	Southern Bellflowers	Aluminum nosepiece, Polyester & Polyurethane earloop, Flat-knitted earloop; "Medicom" embossed at the bottom of mask	Non-Sterile	Medical Device Class 2	Medical Device Class 1			

AMD Medicom Inc. | 2555 Chemin de l'Aviation, Montreal, Quebec, Canada H9P 2Z2

APP1 06.03-45EN_2M Date: 14-Sep-2021 Page 1 / 1